


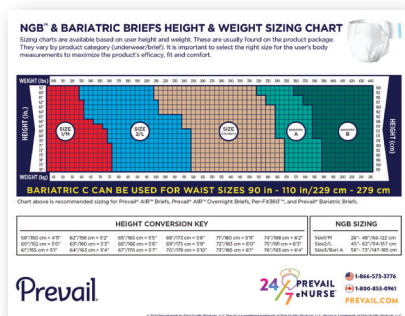


PRODUCT SELECTION

SELECT THE MOST APPROPRIATE PRODUCT BASED ON LEVEL OF INCONTINENCE AND MOBILITY



Choose the flow that best describes your urinary incontinence	MOBILE 	NON MOBILE 	RECOMMENDED PRODUCT FOR YOU:	
LIGHT FLOW A few drops? 	✓			BLADDER CONTROL PADS & MALE GUARDS Prevail has light absorbency product options designed specifically for women and men. Bladder Control Pads are intended for women, which are available in a variety of absorbencies. Male Guards are intended for the male anatomy. These products are designed for bladder leaks and those who are active and mobile.
MODERATE FLOW More than a few drops? 	✓			ADULT PROTECTIVE UNDERWEAR OR PANT LINERS Prevail's Protective Underwear is available in both unisex and gender specific options providing dignity and to help promote independence. These products are designed for heavy leaks, streams and surges for those who are mobile and able to transfer or walk to the toilet. Protective Underwear would not be appropriate for fecal incontinence.
		✓		Briefs are appropriate for those who are immobile.
HEAVY FLOW Steady flow? 	✓	✓		ABSORBENT BRIEFS Prevail Adult Briefs are designed to provide superior protection for heavy bladder leaks and/or fecal incontinence. Adult Briefs are ideal for those who are not active, immobile or require additional night time protection.

USE PREVAIL® SIZING TOOLS TO DETERMINE THE CORRECT SIZE








SCAN QR CODE
TO SIGN UP OR VISIT THE
PREVAIL PROFESSIONAL PORTAL
TO DOWNLOAD DIGITAL
HEIGHT WEIGHT CHARTS



 1-866-573-3776
 1-800-853-0961
PREVAIL.COM

PRODUCT SELECTION

TYPES OF INCONTINENCE

TYPE	WHAT IS IT?	SUGGESTED PRODUCT
 STRESS	Involuntary loss of drips or very small amounts of urine	Bladder Control Pads or Male Guards
 URGE	Involuntary loss of urine that occurs after the sensation to void is present, may occur more than 8 times in 24hrs	Bladder Control Pads, Male Guards or Protective Underwear, Adult Briefs
 MIXED	Combination of Urge and Stress Incontinence	Bladder Control Pads, Male Guards or Protective Underwear, Adult Briefs
 OVERFLOW	Leakage of urine when the bladder reaches a level of fullness causing overflow	Protective Underwear or Adult Briefs
 FUNCTIONAL	Cognitive or physical limitation that impacts continence	Depends on the physical function of the individual as to what type of product used, not completely dependent on the amount of leakage

FACTORS TO CONSIDER WHEN CHOOSING ABSORBENT PRODUCTS

QUALITY

Disposable incontinence products vary in quality depending on the manufacturer. The use of high quality materials and advanced moisture-wicking capabilities are essential factors in ensuring both the comfort and effectiveness for the wearer.

SIZE AND FIT

After selecting the appropriate type of product, it is important to determine the correct size. A larger size does not equate to a better fit and more absorbent. Prevail® protective underwear and adult briefs are designed to fit closely to the body, ensuring both protection and comfort for the wearer.

Prevail® sizing charts use height and weight to recommend the size for our products.

PREVAIL® PRODUCT SELECTION

Choosing the right incontinence product is based on an individual's specific type of incontinence, level of leakage, and mobility. To find the most suitable product to meet these needs, consult the Prevail® Product Selection Guide or contact a Prevail® eNurse™.

DOUBLE PADDING

Prevail® products are intended for individual product use. If the wearer is experiencing leakage, please refer to the Product Selection Guide and Sizing recommendation as double padding is NOT recommended for added absorbency protection. If you need further assistance, our eNurse team is readily available to provide additional support.



 1-866-573-3776
 1-800-853-0961
PREVAIL.COM